




# Dräger DrugCheck 3000

## Drug Testing Device

---

Use the DrugCheck 3000 to find out within minutes whether a person has recently consumed certain drugs. The compact and quick saliva (oral fluid)-based drug test delivers reliable results affordably and easily. The non-invasive drug test kit does not require electricity and can therefore be used anywhere.

# Dräger DrugCheck 3000

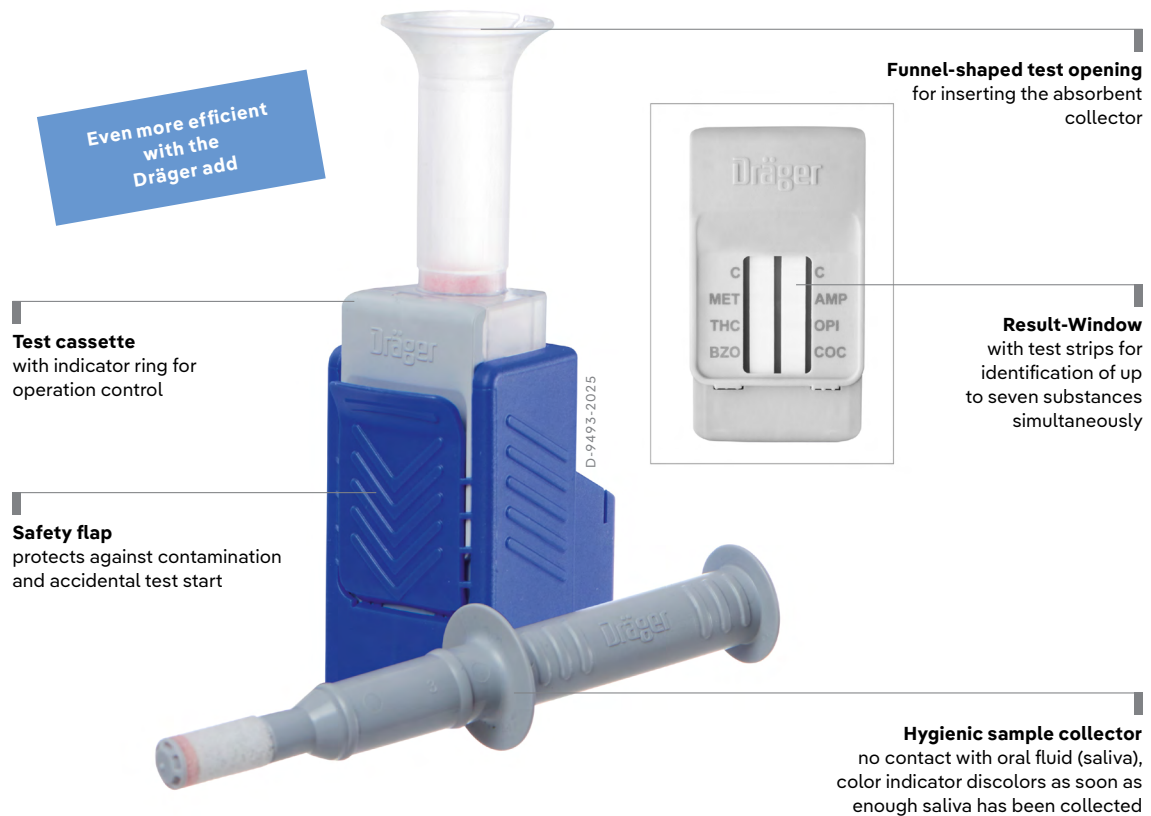

# Benefits

## Drug test – obtain samples easily and safely

The DrugCheck 3000 consists of two components: an hygienic sample collector for obtaining a saliva sample, and a test cassette for the analysis. The test cassette contains the buffer liquid and a window with two test strips, which displays the control and test lines. The drug test itself is performed in three easy steps: swab the saliva sample, shake the test kit, wait for a brief incubation period, and then start the test. A colour indicator on the swab disappears as soon as it has absorbed enough oral fluid for a test.

In case of a negative Test result, a line will appear alongside the respective substance class (drug). As soon as the lines appear in the result window, you can read the results. This means that none of the target substance was detected in the sample. If a line fails to appear next to one of the substance classes, then the result for this substance is positive.

## Fast and sensitive drug analysis

With the DrugCheck 3000 test kit, you test the subjects for up to seven substance classes simultaneously. Depending on the version, the test can detect the following 7 substance classes: Cocaine, Opiates, Amphetamine, Methamphetamine/Designer Drugs (ex. ecstasy, MDMA), benzodiazepines (e.g. in medications), oxycodone and cannabis (THC). Of all the substance classes listed, cannabis is the drug consumed most frequently and also the most difficult of all compounds to identify (THC = delta-9-tetrahydrocannabinol). This is why the DrugCheck 3000 was optimised to detect THC, and now offers two measurement options: depending on the chosen waiting time (pre-incubation) of the test there is a fast option with a higher or sensitive option with a lower THC cut-off. The entire drug test only takes 3 – 5 minutes.

## Unambiguous on-the-spot drug screening

Its compact pocket sized design makes the DrugCheck 3000 easy to transport. It can be made ready for use quickly and easily, and is suitable for flexible use applications such as roadside checks and testing in safety-relevant workplaces. The DrugCheck 3000 is intended for forensic applications, as well as for use by laypersons and in the workplace. Observe the local and national regulations for the respective intended use. The test kit has no electrical parts, which makes the drug tester safe to use in hazardous areas and even as point-of-care-testing (POCT).

## Additional benefits

Dräger has decades of experience in alcohol measuring equipment and drug detection methods. For the DrugCheck 3000, Dräger employed the testing principle of the proven DrugTest 5000 system, which reliably detects even minute traces of THC. The disposable DrugCheck 3000 test kit cannot be manipulated and is particularly hygienic to use.

# Benefits


## Go digital: drug detection just got even more convenient and efficient with

### Dräger add

Simplify your drug and/or alcohol testing documentation process with Dräger add. Scan, document and archive your test results in just a few steps using the mobile app. The application can be tailored to your needs and processes. In addition, it is possible to view statistics on recorded test data, export customisable PDF test reports and generate printouts using the Mobile Printer BT. The Dräger add app is available for Android and iOS operating systems. The test data can be automatically uploaded from the mobile device and stored centrally for authorised users to access anytime, anywhere. The web application provides management options for all test data, including analytical tools such as a heat map and various charts and filter options. More detailed information about specific tests and their status can be found on the details page providing a comprehensive view of test operations.

## Details

D-49687-2015

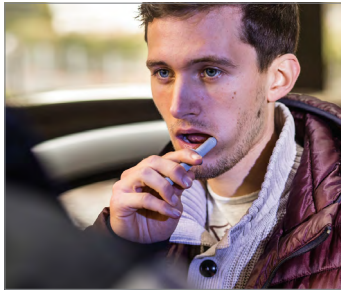

1. Take a saliva sample using the sampler

D-49707-2015



2. Insert the sampler into the funnel opening

D-49724-2015



3. Shake the test kit

D-15447-2017



4. Start the test analysis

D-15442-2017



5. Read off the test results

D-10311-2025



6. Get more efficient with our modular software solution: "Dräger add"

## Services

D-9492-2025



### Dräger add download link

Scan the QR code to download Dräger add for free directly from the Google Play Store or Apple App Store.

## Related Products

D-54720-2012



### DrugTest 5000

No pipetting, no drips, no timing: With the DrugTest 5000, a drug test is carried out simply and quickly. The collected saliva sample can be analysed immediately – for an accurate result on the spot.

D-6296-2009



### SSK 5000

Ideal for surface drug testing applications, the SSK 5000 sampling system is quick and easy to use. When used with the DrugTest 5000, the sample can be transferred to a test cassette and screened for possible drug contamination. The sample can also be stored and transported for further testing.

ST-13225-2007



### DCD 5000

Simple, fast and secure detection of intoxicants and drugs: The DCD 5000 does not only enable an easy collection of saliva (oral fluid), you can also store the samples and send them to the laboratory for analysis – without the sample being falsified along the way.

D-15278-2017



### SCK

The SCK confirmation set contains two samplers and stabilising components (isopropanol) as well as documentation forms and shipping materials, ensuring a chain of custody through clear documentation and secure shipment of samples to the lab.

# Technical Data

## Test for detecting drugs in oral fluid

Dimensions (W x H x D)	32 x 111 x 57 mm 1.26 x 4.37 x 2.24 inches
Weight	< 30 g (0,07 lb)
Operating range	Operation: 5 °C to 40 °C (41 °F to 104 °F) at 5 % to 95 % relative humidity
Storage / transportation	4 °C to 25 °C (39 °F to 77 °F)
Time of measurement	Fast measurement: analysis < 3 minutes Sensitive measurement: analysis < 5 minutes
Selecting the measurement mode	Depending on the selected pre-incubation time and the desired THC cut-off fast or more sensitive reading.

# Ordering Information

DrugCheck 3000 Test for detecting drugs in oral fluid	Description	Ordering no.
DrugCheck 5-Panel	Packaging unit 20 kits	8325580
The DrugCheck 5-Panel detects the following substances:		
<ul style="list-style-type: none"> <li>• Cocaine (COC)</li> <li>• Opiates (OPI)</li> <li>• Cannabis (THC)</li> <li>• Amphetamines (AMP)</li> <li>• Methamphetamines (MET)</li> </ul>		
DrugCheck 6-Panel BZO	Packaging unit 20 kits	8327960
The DrugCheck 6-Panel Benzodiazepines detects the following substances:		
<ul style="list-style-type: none"> <li>• Cocaine (COC)</li> <li>• Opiates (OPI)</li> <li>• Cannabis (THC)</li> <li>• Amphetamines (AMP)</li> <li>• Methamphetamines (MET)</li> <li>• Benzodiazepines (BZO)</li> </ul>		
DrugCheck 6-Panel BZO Single-Pack	Packaging unit 1 kit	3727400
The DrugCheck 6-Panel Benzodiazepines detects the following substances:		
<ul style="list-style-type: none"> <li>• Cocaine (COC)</li> <li>• Opiates (OPI)</li> <li>• Cannabis (THC)</li> <li>• Amphetamines (AMP)</li> <li>• Methamphetamines (MET)</li> <li>• Benzodiazepines (BZO)</li> </ul>		
DrugCheck 6-Panel OXY	Packaging unit 20 kits	3711990
The DrugCheck 6-Panel Oxycodone detects the following substances:		
<ul style="list-style-type: none"> <li>• Cocaine (COC)</li> <li>• Opiates (OPI)</li> <li>• Cannabis (THC)</li> <li>• Amphetamines (AMP)</li> <li>• Methamphetamines (MET)</li> <li>• Oxycodone (OXY)</li> </ul>		
DrugCheck 7-Panel	Packaging unit 20 kits	3722343
The DrugCheck 7-Panel detects the following substances:		
<ul style="list-style-type: none"> <li>• Cocaine (COC)</li> <li>• Opiates (OPI)</li> <li>• Cannabis (THC)</li> <li>• Amphetamines (AMP)</li> <li>• Methamphetamines (MET)</li> <li>• Oxycodone (OXY)</li> <li>• Benzodiazepines (BZO)</li> </ul>		



Not all products, features, or services are for sale in all countries. Trademarks mentioned herein are the property of its respective owner. Trademarks may be owned by Drägerwerk AG & Co. KGaA (Dräger) or its affiliates in certain countries and not necessarily in the country in which this material is released. Visit [www.draeger.com/trademarks](http://www.draeger.com/trademarks) for the current status of Dräger's trademarks.

**Corporate Headquarters**

Drägerwerk AG & Co. KGaA  
Moislinger Allee 53–55  
23558 Lübeck, Germany  
■ [www.draeger.com](http://www.draeger.com)

**Region Europe****Dräger Safety AG & Co. KGaA**

Revalstraße 1  
23560 Lübeck, Germany  
☎ +49 451 882 0  
☎ +49 451 882 2080  
✉ [info@draeger.com](mailto:info@draeger.com)

**Region Middle East, Africa****Dräger Safety AG & Co. KGaA**

Branch Office  
P.O. Box 505108  
Dubai, United Arab Emirates  
☎ +971 4 4294 600  
☎ +971 4 4294 699  
✉ [contactuae@draeger.com](mailto:contactuae@draeger.com)

**Region Asia Pacific****Dräger Singapore Pte. Ltd.**

61 Science Park Road  
The Galen #04-01  
Singapore 117525  
☎ +65 6872 9288  
☎ +65 6259 0398

**Region Central and South America****Dräger Safety do Brasil Ltda.**

Al. Pucuruí, 51/61 – Tamboré  
06460-100 Barueri, São Paulo Brasil  
☎ +55 (11) 4689-4900  
✉ [relacionamento@draeger.com](mailto:relacionamento@draeger.com)



Locate your Regional Sales  
Representative at:  
[www.draeger.com/contact](http://www.draeger.com/contact)